

Kaolin Suspension

helena
Biosciences Europe

REF 53765



Helena Biosciences Europe, Queensway South, Team Valley Trading Estate, Gateshead, Tyne and Wear, NE11 0SD, United Kingdom

Tel: +44 (0)191 482 8440

Fax: +44 (0)191 482 8442

Email: info@helena-biosciences.com

Web: www.helena-biosciences.com

HL-2-3234P 2016/06 (1)

Kaolin Suspension Instructions for use

en

INTENDED PURPOSE

The Kaolin Suspension kit is intended for carrying out clot based haemostasis assays.

Kaolin Suspension is used to reconstitute Thrombin where instrument methodology indicates Kaolin reconstitution of the Thrombin reagent is required for the use in Clauss Fibrinogen assays. The instructions supplied with the analyser and/or the test procedure should be followed for use.

WARNINGS AND PRECAUTIONS

The reagents contained in this kit are for *in vitro* diagnostic use only – DO NOT INGEST. Wear appropriate personal protective equipment when handling all kit components. Refer to the product safety declaration for the link to appropriate hazard and precautionary statements where applicable. Dispose of components in accordance with local regulations.

COMPOSITION

Component	Content	Description	Preparation
Kaolin Suspension	1 x 100 mL	Each bottle contains 100 mL of Kaolin Suspension at 0.5 g/L.	Mix vigorously immediately before use. Ensure no sedimentation is visible.

Each kit contains Instructions For Use.

ITEMS REQUIRED BUT NOT PROVIDED

Helena Biosciences Europe supply the following kits available for use with this product:

REF 5376, 5376R	Clauss Fibrinogen 100
REF 5374	Clauss Fibrinogen (Thrombin Only)
REF 5378	Clauss Fibrinogen (Thrombin Only)
REF 5379	Fibrinogen Calibrator
REF 5375	Owren's Buffer
REF 5375R	Imidazole Buffer

STORAGE, SHELF-LIFE AND STABILITY

Unopened bottles are stable until the given expiry date when stored under conditions indicated on the bottle label. Avoid contamination and discard if any particulate matter indicative of contamination is observed. Store at 2 – 8°C once opened.