Calcium Chloride

helena
Biosciences Europe

REF 5386

Helena Biosciences Europe, Queensway South, Team Valley Trading Estate, Gateshead, Tyne and Wear, NE11 0SD, United Kingdom

Tel: +44 (0)191 482 8440

Fax: +44 (0)191 482 8442

Email: info@helena-biosciences.com

Web: www.helena-biosciences.com

HL-2-0655P 2016/01 (3)

Calcium Chloride Instructions for use

en

INTENDED PURPOSE

The Calcium Chloride kit is intended for carrying out clot based haemostasis assays.

WARNINGS AND PRECAUTIONS

The reagents contained in this kit are for *in vitro* diagnostic use only – DO NOT INGEST. Wear appropriate personal protective equipment when handling all kit components. Refer to the product safety declaration for the link to appropriate hazard and precautionary statements where applicable. Dispose of components in accordance with local regulations.

COMPOSITION

Component	Content	Description	Preparation
Calcium Chloride: 0.025M	10 x 10 mL	Each vial contains 0.025 M calcium chloride solution. The reagent is a clear, colourless solution.	The reagent is ready for use as packaged.

Each kit contains Instructions For Use.

STORAGE, SHELF-LIFE AND STABILITY

Unopened reagents are stable until the given expiry date when stored under conditions indicated on the vial or kit label.

Calcium Chloride: 0.025M	Opened vials are stable for 1 month at +2 –+8°C or 5 days at +15 –+30°C. Particulate contamination, cloudiness or a change in expected performance may indicate product deterioration.
--------------------------	--

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